

## ▶ footwear measuring guide

kids feet grow fast,  
measure every time you order!

## ▶ accuracy check


Before using this sizing chart, be sure it has printed to scale by measuring the box with a ruler. The height of the box should be 9". If the measurement is not correct, please check your print settings and make sure the page scaling option is set to 'none'. If problems persist, please feel free to give our customer service representatives a call for assistance.

## ▶ instructions

For an accurate fit, please follow these guidelines carefully:

- Measure your child's feet with their socks on.
- Place this sizing chart on a hard flooring surface, and have you child place their heel at the bottom of the chart, where indicated.
- Remember to always measure both feet and order the shoe that fits the larger of the two.

place heel here ▶

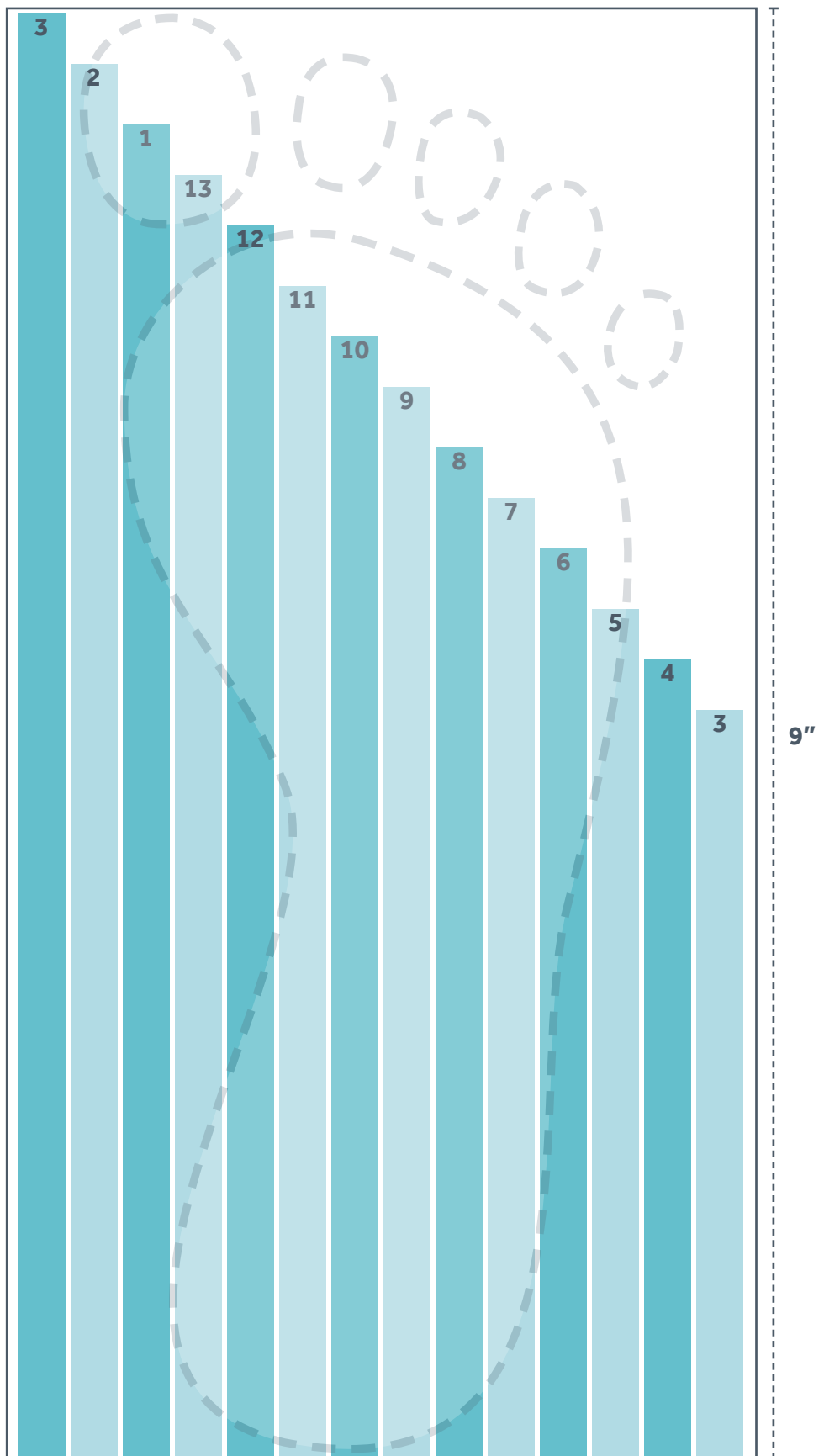



## Sizing Chart

To ensure an accurate fit, please reference this sizing chart and the measuring guide before ordering your pair of Surestep shoes.

SIZE	FOOT LENGTH		#6 MEASUREMENT	
	Inches	Centimeters	Inches	Centimeters
3	4 <sup>5</sup> / <sub>8</sub> "	11	2 <sup>3</sup> / <sub>4</sub> " - 3"	7-8
4	4 <sup>7</sup> / <sub>8</sub> "	12	2 <sup>7</sup> / <sub>8</sub> " - 3 <sup>1</sup> / <sub>4</sub> "	7.5-8.5
5	5 <sup>1</sup> / <sub>4</sub> "	13	3 <sup>1</sup> / <sub>8</sub> " - 3 <sup>1</sup> / <sub>2</sub> "	8-9
6	5 <sup>5</sup> / <sub>8</sub> "	14	3 <sup>3</sup> / <sub>8</sub> " - 3 <sup>3</sup> / <sub>4</sub> "	8.5-9.5
7	5 <sup>15</sup> / <sub>16</sub> "	15	3 <sup>5</sup> / <sub>8</sub> " - 4"	9.5-10.5
8	6 <sup>1</sup> / <sub>4</sub> "	16	3 <sup>7</sup> / <sub>8</sub> " - 4 <sup>1</sup> / <sub>4</sub> "	10-11
9	6 <sup>5</sup> / <sub>8</sub> "	17	4 <sup>1</sup> / <sub>8</sub> " - 4 <sup>1</sup> / <sub>2</sub> "	10.5-11.5
10	6 <sup>15</sup> / <sub>16</sub> "	17.5	4 <sup>3</sup> / <sub>8</sub> " - 4 <sup>3</sup> / <sub>4</sub> "	11.5-12.5
11	7 <sup>1</sup> / <sub>4</sub> "	18	4 <sup>5</sup> / <sub>8</sub> " - 5"	12-13
12	7 <sup>5</sup> / <sub>8</sub> "	19	4 <sup>7</sup> / <sub>8</sub> " - 5 <sup>1</sup> / <sub>4</sub> "	12.5-13.5
13	7 <sup>15</sup> / <sub>16</sub> "	20	5 <sup>1</sup> / <sub>8</sub> " - 5 <sup>1</sup> / <sub>2</sub> "	13-14
1	8 <sup>1</sup> / <sub>4</sub> "	21	5 <sup>3</sup> / <sub>8</sub> " - 5 <sup>3</sup> / <sub>4</sub> "	13.5-14.5
2	8 <sup>5</sup> / <sub>8</sub> "	22	5 <sup>5</sup> / <sub>8</sub> " - 6"	14.5-15.5
3	8 <sup>15</sup> / <sub>16</sub> "	23	5 <sup>7</sup> / <sub>8</sub> " - 6 <sup>1</sup> / <sub>4</sub> "	15-16