

INDY 2 STAGE

*You are in control! Combine SMO and AFO styles to create unique solutions that best serve your patients. Start by choosing either **solid ankle** or **hinged** and continue modifying as needed.*

» Solid Ankle

Additional fabrication options available:
PLS or Floor Reaction

Posterior Leaf Spring (PLS)

Corrugation options:

- Plastic
- Rope
- Carbon fiber reinforcement

Floor Reaction

Optional carbon fiber reinforcement



» Hinged

Snapstop only

Choice of hinge:

- Surestep Free Motion
- Surestep Dual Adjustable
- Tamarack Free Motion
- Tamarack Dorsi Assist



Modifications

AFO

- ☐ Full-Length Footplate
- ☐ Surestep Trim Lines
- ☐ Removable Tibial Shell
- ☐ Nighttime Stretching Straps
- ☐ Open Heel AFO
- ☐ Adjustalift
- ☐ Toe Plateaus



» TIPS & TRICKS

Add an instep strap! A strap across the dorsum can be added to the AFO to help keep the SMO in place.

To reduce bulk in the toe box, consider trimming either the footplate or the walls of the AFO.

Modifications

SMO

- ☐ Toe Walking Extension
- ☐ Open Heel
- ☐ Houdini Straps
- ☐ Reverse Trim Line



INDY 2 STAGE

» Combine any SMO with any AFO to customize your Indy 2 Stage

